User Manual

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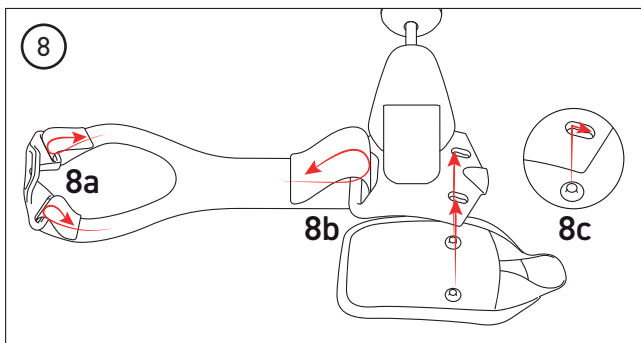
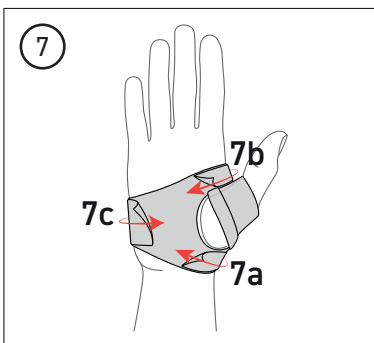
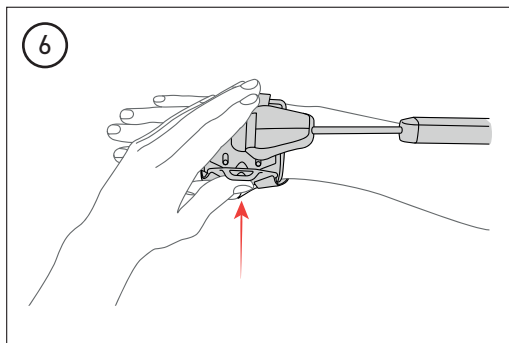
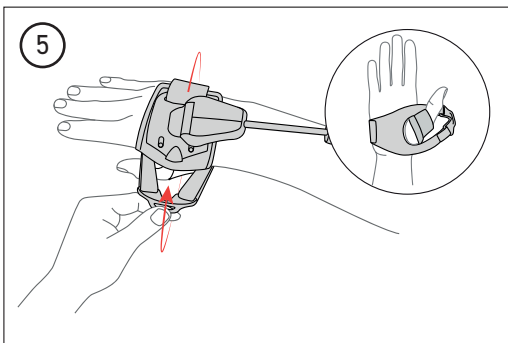
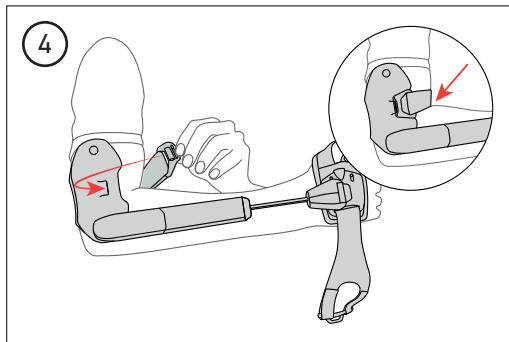
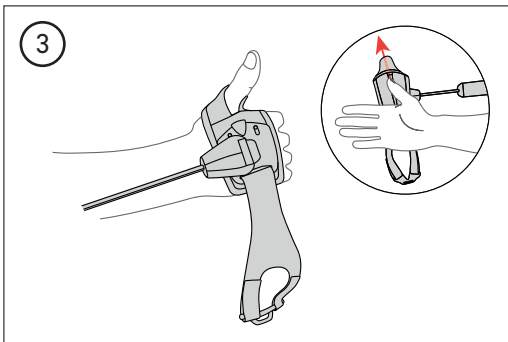
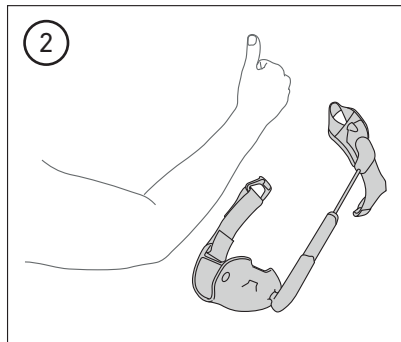
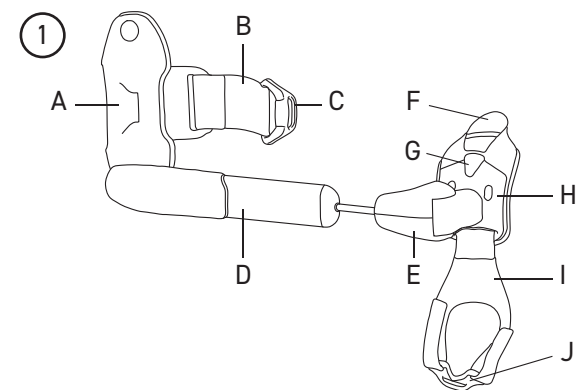
English

Introduction

The STIL Orthosis is an elbow-hand orthosis for the alleviation of lower arm and wrist tremors. It reduces tremors by means of mechanical damping. The orthosis' performance in suppressing tremor may vary from person to person.

Intended use

The STIL Orthosis is intended to be worn on an arm affected by invalidating arm tremors predominantly involving wrist flexion-extension and forearm pronation-supination. It supports the user while performing activities of daily living, such as eating or drinking. This orthosis is for a single patient - multiple use.

Indications

The orthosis is intended for users of 18 years of age or older, diagnosed with essential tremor. Suitability of the orthosis for the user must be evaluated by a trained healthcare professional. The orthosis must be installed and fitted by a trained healthcare professional.

Contraindications Δ

Consult a physician before using the orthosis if: you are a blood donor; your arm or shoulder has a history of muscle or tendon injury or irritation; or if you have skin irritation, swelling, redness, or other skin conditions; or if you have osteoporosis or muscle atrophy.

Side effects

The use of the orthosis may cause muscle fatigue, skin irritations, or discomfort in the hand or elbow. In case of pain, skin lesions, cuts or other side effects, stop using the orthosis and contact your healthcare professional.

Warnings and important notes Δ

- Consult your physician before discontinuing any tremor-related medication.
- Consult your healthcare professional before using the orthosis during sleep.
- The locking mechanisms of the orthosis contain magnets: keep them away from pacemakers or implantable cardioverter defibrillators.

- If you experience pain or reduction of product performance, stop using the orthosis and contact your healthcare professional.
- Do not wear the orthosis on damaged skin (such as open wounds, irritated tissue, burned areas, infected tissue).
- The orthosis partially limits mobility. Do not perform dangerous tasks (such as driving a vehicle, or handling sharp objects) while wearing the orthosis.
- Wash fabrics components before use and at least once per month.
- The orthosis is designed for a maximum lifetime of 2 years.
- Keep the orthosis away from dust, dirt, and sand.
- Keep the orthosis dry.
- Do not use the orthosis over watches, bracelets, glucose sensors, or other wearable devices.
- The orthosis may damage clothing.
- Do not wear the orthosis under tight clothing.

Fitting instructions

Before using the orthosis for the first time, practice putting it on and taking it off with a trained healthcare professional. When performing the following instructions, refer to the overview image to locate the parts named in the text.

- A** elbow cuff
- B** elbow strap
- C** elbow clip
- D** slider
- E** wrist element
- F** thumb strap
- G** hook
- H** hand cuff
- I** hand strap
- J** hand clip

Putting on the orthosis

- 1 Open the elbow clip (C) and hand clip (J) as shown in the overview image;
- 2 Raise your thumb and place the orthosis on the table on the outside of your arm;
- 3 Slide the thumb strap (F) over your thumb;
- 4 Place the elbow element (A) as low as possible around the upper arm and close the elbow strap (B) with the elbow clip. Tighten the strap so that it fits firmly around the arm;
- 5 Insert your thumb through the hand strap (I);

- 6 Push the hand clip over the hook (G);
- 7 Tension the hand strap by adjusting the hook-and-loop. Start with the strap under the thumb (7a), then the strap between the thumb and index finger (7b), and finish with the strap at the little finger end (7c).

Taking off the orthosis

- No need to undo the hook-and-loop to remove the orthosis;
- Open the hand-clip (J);
- Open the elbow-clip (C);
- Remove the orthosis from the arm and hand.

Fitting the fabrics

- 8a Pass the two narrow straps (7a) (7b) of the hand strap through the holes of the hand clip as shown;
- 8b Then insert the wide (7c) strap of the hand strap through the hand cuff, making sure to close the hook-and-loop at the top;
- 8c Then insert the two caps of the thumb strap through the small holes in the hand cuff and slide the thumb strap to the right.

Cleaning instructions

It is recommended to regularly clean the orthosis to maintain adequate levels of hygiene.

Textile parts

- Remove the textile parts from the orthosis;
- Fasten all hook-and-loop closures;
- The textile parts can be washed at temperatures up to 30°C, on delicate cycle or by hand;
- Air-dry and do not expose to direct heat sources such as stoves or radiators.

Note: Do not tumble dry, iron, bleach, or wash with fabric softener.

Other parts

- Clean plastic and metal components with a damp cloth.

Maintenance

The product requires no maintenance if handled and cared for properly.

Disposal

In order to dispose of the orthosis, adhere to local regulations and instructions.

Report serious incidents

As imposed by local regulations, you are requested to immediately report any serious incident involving the

use of this medical product to both manufacturer and relevant authorities. Our contact details can be found on the back of this brochure.

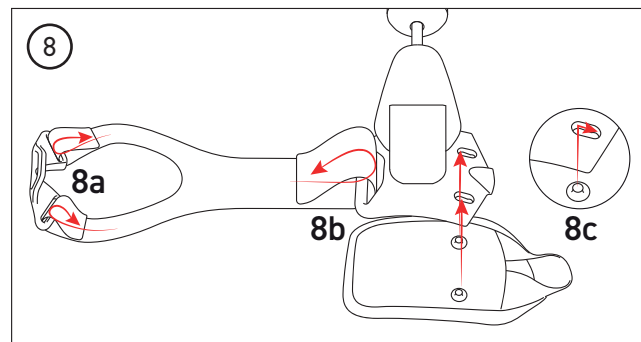
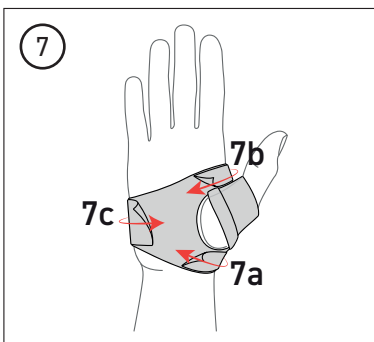
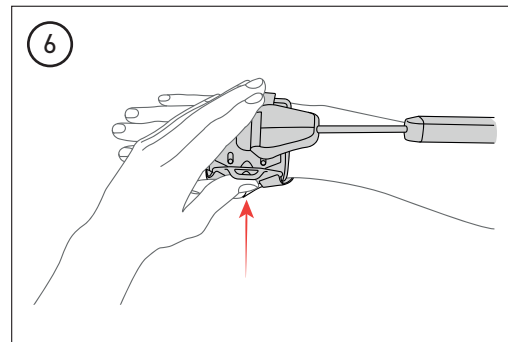
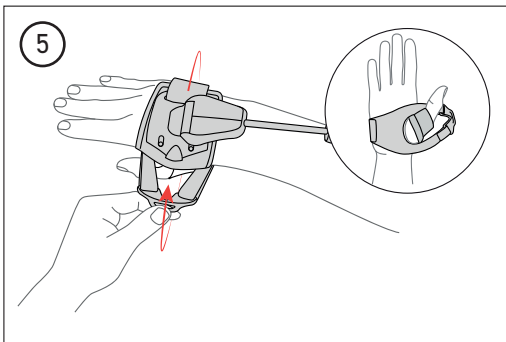
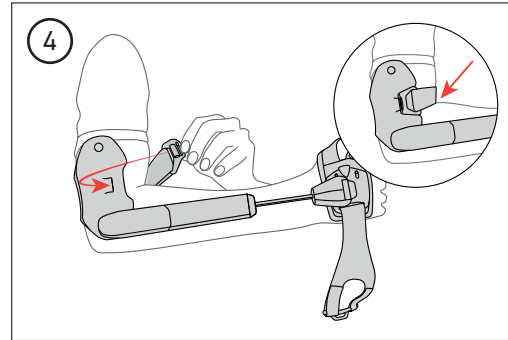
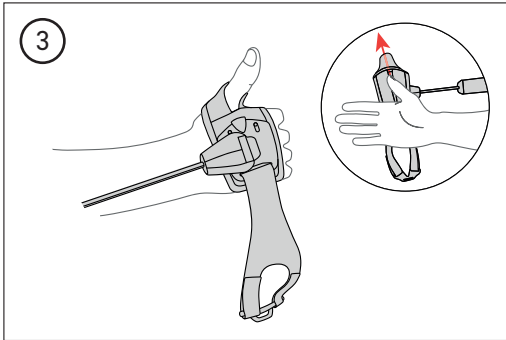
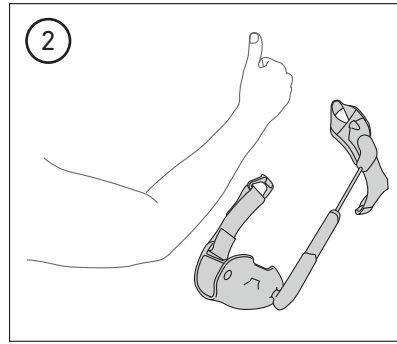
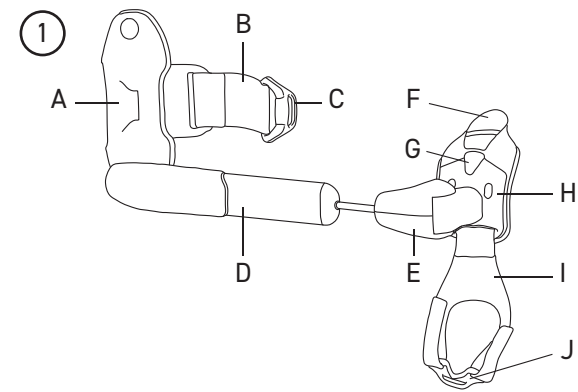
Liability

STIL does not assume liability for the following:

- Product repairs after the warranty period;
- Improper use or unauthorized modification of the product;
- Product use outside of intended or recommended use condition, or against provided indications and contraindications.

Materials

Textile parts contain: Nylon (polyamide), polyester, elastane, polyurethane, elastodiene.

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